



GOLDEN
Passport[™]
HI LOW ADJUSTABLE BED



Connect the
Passport Hi Low bed
easily to your smart
devices with the
Bluetooth Adapter

goldentech.com

Getting Started

Comfort Enhancements



1. On your smart phone or tablet download the 'Comfort Enhancements' app
2. Make sure the Bluetooth feature is turned on your smart device
3. Open the 'Comfort Enhancements App' and select the 'Hi Lo Lift' controls for the Passport Hi Low bed
4. In the top right corner of the screen open the Settings menu and click 'Select Device'
5. Select the "OKIN" device
6. Control your Passport bed remotely using the app on your smart phone or tablet

