



GOLDEN  
*Passport*<sup>™</sup>  
HI LOW ADJUSTABLE BED



Connect the  
Passport Hi Low bed  
easily to your smart  
devices with the  
**Bluetooth Adapter**

[goldentech.com](http://goldentech.com)

# Pairing Instructions



1. **App Installation:** Download and install the 'Comfort Enhancements 3' app on your preferred smart device.
2. **Setup Bluetooth:** Turn on the Bluetooth feature to connect to the Bluetooth adapter seamlessly.
3. **App Navigation:** Open the app and swipe left to locate the Hi Low bed remote option. It will be the last option.
4. **Power Reset:** Follow the on-screen instructions to unplug your bed for 2 minutes, then reconnect the power.
5. **Pairing Initiation:** Press the head and foot buttons together on the Hi Low bed remote twice to activate pairing mode. Look for the flashing lights under your bed as an indicator.
6. **Device Selection:** Choose the "OKIN" device from the list on your smart device. Listen for the jingle from the control box to confirm successful pairing.
7. **Effortless Control:** Enjoy controlling your bed's settings remotely, adjusting to your comfort preferences with ease.

